

CHEMICAL INDICATOR LABELS

DATA SHEET

CHEMICAL INDICATOR LABELS

REF CL801


Chemical Indicator Labels for Air Plasma Sterilization® are used as EXTERNAL pack or pouch process indicators to verify exposure to sterilant gases produced by the Air Plasma Sterilization® process. These chemical indicator labels consist of a nonporous plastic material printed with a chemical indicating ink and a pressure sensitive adhesive backing. Chemical Indicator Labels can be placed on the exterior of packs or pouches to help differentiate processed items from unprocessed items. **The chemical indicating ink turns from RED to YELLOW when exposed to sterilant gases.**

TECHNICAL SPECIFICATIONS


Product Code	CL801
Process	Air Plasma Sterilization® only
Dimensions	3 in. x 1 in. (76.2 mm x 25.4 mm)
Chemical Indicator	Initial color: RED Processed color: YELLOW
Usable life	Single use. DO NOT REUSE.
Shelf life	Indefinite when stored in original packaging
Storage	Away from heat, moisture, and oxidative gases
Disposal	No restrictions, dispose with ordinary waste
Packaging	500 labels per roll



FOR USE WITH AIR PLASMA STERILIZERS ONLY

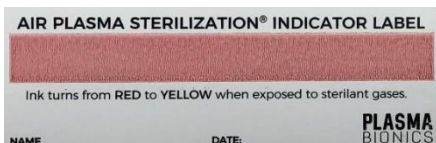

UNEXPOSED

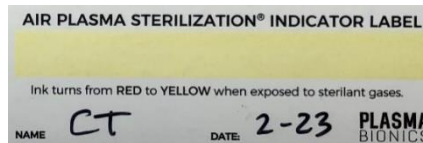

EXPOSED

INTERPRETATION INSTRUCTIONS

Please note that chemical indicators do not ensure sterility, only that appropriate concentrations of sterilant gases were present. Only biological indicators should be used to confirm sterility. **Properly exposed chemical indicators will change from RED to YELLOW.**

TREAT ALL INDICATORS THAT ARE NOT COMPLETELY YELLOW AS PROCESS FAILURES.



plasmabionics.com

sales@plasmabionics.com

+1 (405) 939 0676