

PLASMA SHEETS

DATA SHEET


PLASMA SHEETS

REF S201

The key component to the Air Plasma Sterilization® process is the proprietary Plasma Sheet. Plasma is produced on the surface of the plasma sheet inside the sterilization chamber, converting air into highly oxidative sterilant gases. Plasma sheets wear out over time and must be replaced after 15 sterilization cycles. If a sheet has been damaged or has reached its maximum number of uses, an error will appear on the sterilizer touchscreen display once a sterilization cycle has been started. The sterilizer touchscreen display will prompt the user to replace the sheet when necessary. This robust, durable consumable has an indefinite shelf life and has no special disposal restrictions.

TECHNICAL SPECIFICATIONS

Product Code	S201
Process	Air Plasma Sterilization® only
Dimensions	11.5 in. x 7 in. (29.2 cm x 17.8 cm)
Usable life	15 sterilization cycles
Shelf life	Indefinite when stored in original packaging
Storage	Away from heat and moisture
Disposal	No restrictions, dispose with ordinary waste
Packaging	4 plasma sheets per box

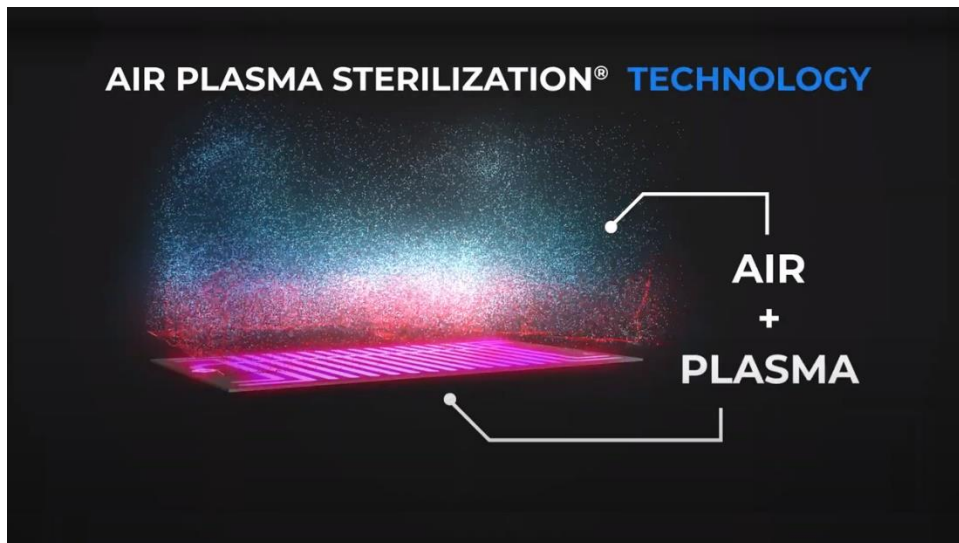

plasmabionics.com
sales@plasmabionics.com
+1 (405) 939 0676